

Lipocosm Lipografter™ Sterilization

Product Identification:

Name: Lipografter™
Supplier: Lipocosm, LLC
Physical State: Devices – non-sterile supplied
Storage Conditions: Room Temperature
Safety Precautions: None

Product Description:

The components of the LipoCosm Lipografter™ are as follows:

- KVIC Syringe
- AT Valve
- Silicone tubing
- Male luer cap, male luer to barb connector and female luer to barb connector

Sterilization Process:

The qualified sterilization process is standard steam autoclave and drying cycle.

Sterilization Cycle:

A sterilization cycle with a temperature of 130° C (266° F) and a time of 20 minutes is considered qualified to support a SAL of 10⁻⁶ for use with the Lipografter™.

Lipocosm, LLC
305-361-5771

Sterilization Validation Test Report is available on request